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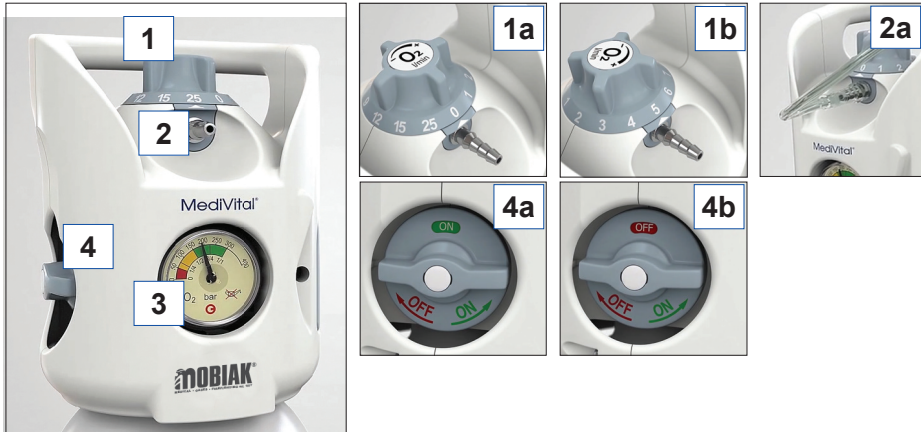
QUICK REFERENCE GUIDE

BEFORE USE

Please check the **Combination Valve** for any signs of damage or contamination. Also check that the **gas Pressure Indicator** **3** is showing sufficient gas for your needs. Please note that the Pressure Indicator **3** always displays the current gas pressure even when the Shut Off Valve **4** is in the off position.

Checking the functions of the Combination Valve:

- Check that the Flow Control Knob **1** is set at 0.
- Open the Shut Off Valve **4** by slowly turning until ON is fully displayed in the window **4a**.
- Check for leaks by listening to the device.
- Turn the Flow Control Knob **1** clockwise to position 1 and check that gas is flowing through the outlet **2**. Continue to turn the Flow Control Knob **1** to check that gas flows from each of the different settings.
- Reset Flow Control Knob **1** to 0.

USING THE COMBINATION VALVE

- Check that the Flow Control Knob **1** is set at 0.
- Connect the accessories (cannula or mask) to the Flow Outlet **2a**.
- Open the Shut Off Valve **4** by slowly turning until ON is fully displayed in the window **4a**.
- Set the Flow Control Knob to the prescribed position. Check that the Flow Control Knob is aligned correctly as in example **1b**.

Before connecting any accessory to the Flow Outlet make sure that the patient is not connected to the device.

Open Shut Off Valve SLOWLY!

Always ensure that the Flow Control Knob has been correctly set and not placed between settings.

Do not try to apply excessive force on the Flow Control Knob!

The correct setting of the Flow Control Knob must be advised by your doctor.

AFTER USE

- Fully turn off the shut off valve by turning the handwheel clockwise until a firm resistance to rotation is reached **4b**.
- If there are any medical devices connected to the outlet, these should be vented.
- Disconnect any medical devices from the Flow Outlet **2a**.
- Set the Flow Control Knob to 0 **1a**.

QUICK REFERENCE GUIDE

SAFETY INSTRUCTIONS

KEEP THE PRODUCT AND ITS ASSOCIATED EQUIPMENT AWAY FROM:

- Heat sources (fire, cigarettes, e-cigarettes...)
- Flammable materials
- All oil and grease; never use hand creams or moisturizers when using or handling the equipment.
- Water
- Dust

The product and its associated equipment must be prevented from tipping over, turning over or falling.

Always maintain oxygen cleanliness standards.

Only use the product and its associated equipment in a well ventilated area.

OPERATING CONDITIONS	
	-20* / +65 °C
	10 / 100 %
	600 / 1200 mbar

Do not use the combination valve in temperatures below -20°C.

Before connecting any accessory or medical device to the Combination Valve, always check that they are fully compatible with connection features and performances of the product.

If the combination valve is damaged or contaminated then do not use it and immediately contact your supplier.

If you have any further questions then please contact our customer service department.

Tel: +30 28210 63222

Email: medicalexports@mobiak.com

Visit our new web site www.mobiakcare.com