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**INTENDED USE:**

The LED X-RAY film viewers are intended to be used as illuminated apparatus for visualizing developed X-ray films. They are suitable to view all sizes of X-RAY films like CT (Computerized Tomography), films, DR (Digital Radiography) films and others, for diagnosing and analyzing in hospitals and clinics.

**CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice
- There is a ± 3%. Deviation for the Dimensions of the Products.

**GENERAL WARNINGS:**

- If you don't read this user manual, it is preferable not to use this product or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Save this document for future reference.
- The screen of X-ray viewer is composed of white polymethyl methacrylate plate, hence do not expose in high temperature so as to prevent damage of the product.
- The plug should be well grounded during installation to prevent electricity leakage.
- The X-ray film viewer is fragile. Ensure extra attention during transportation.
- Do not attempt to use the X-ray film viewer in a humid place.
- The power cable should be unplugged while cleaning, repairing or executing any kind of maintenance of the X-ray film viewers.
- The use of the product in a flammable and explosive gas environment is prohibited.
- When it is not used for a long time, pull out the power plug.

**TECHNICAL SPECIFICATIONS**

Code	Power (W)	Screen Size (cm)	Outer Size (cm)	Average Luminance (lux)	Average Luminance (cd/m <sup>2</sup> )	Thickness (cm)
0804490	35W	36 x 44	44.5 x 58 (56+2)* x 10	≥6300	≥2000	10
0804491	70W	72 x 44	80 x 58.5 (56+2.5)* x 10	≥6300	≥2000	10
0804492	80W	74 x 45	85 x 55 x 2.4	≥13000	≥4000	2.4

\* the thickness of the integrated base is included in the measurement

The screen of X-ray viewer is composed of white polymethyl methacrylate plate

The frame is made by aluminum-alloy

**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

**CLEANING**

Wipe with a dry and clean cloth, The product should not come into contact with water or any other liquid.

Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

**INCIDENT REPORTING:** Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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**GENERAL OPERATING INSTRUCTIONS:**

The products are delivered completely assembled.


Plug in the power supply cable and turn on the ON / OFF switch.

The viewer is lit immediately, and it will reach the normal luminance within 15 minutes.

The X-ray film viewer has a protection circuit. If you need to turn it on again, please turn it on, 10 seconds after you press the off switch.

**For the products 0804490 & 0804491:**

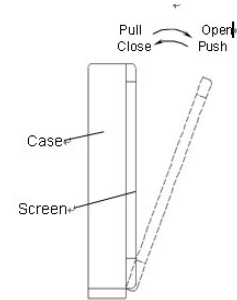Please refer to below sketch for opening and closing the screen in cases of replacement of LED lamps or for repairing purposes.

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**ONLY for the product 0804492:**

Automatic switch (film sensor): The viewing area will be lit when user inserts the film and it will be blacked out when user extracts the film

Individual control function for each connection

Individual dimming function for each connection via rotary switch

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**WARRANTY:**

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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MOBIAK S.A