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INTENDED USE:

A wheelchair ramp is an inclined plane installed in addition to or instead of stairs. Ramps permit wheelchair users, as well as people pushing strollers, rollators, carts, or other wheeled products, for more easily access to a building, to an area with steps, to wheeled vehicles or navigate between areas of different height.

It can be used either as a portable or fixed access ramp according to the model you have.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts.
- Do not use the product for a different purpose from the intended one.
- Do not exceed the weight limit of the safe working load.
- Only allow one person or mobility vehicle to use the ramp each time.
- Only use the ramp very slowly. Do not drive the scooter or power-wheelchair at full speed as this could cause damage and/or injury.

BEFORE EVERY USE:

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects
- Ensure the ramp is folded out and securely positioned before every use.
- Ensure the load on the ramp is approximately central between the left and right sections. Do not lean over the edge during use.
- The ramp can be permanently installed. Attach the ramp using the two holes predrilled at the front of the ramp and suitable fasteners, that not included in the packaging.
- Use the lowest possible inclination of ascent or descent and do not exceed the inclination proposed by the manufacturer of your wheelchair.

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

- Gloves should be worn.
- Remove the evident dirt before to proceed with the most accurate cleaning.
- Use water or mild detergent for the cleaning procedure.
- Wipe with a dry and clean cloth.
- DO NOT leave the product wet.
- Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
- If you need to disinfect the product, use a common, mild disinfectant
- Protect the item from scratches, cuts and punctures.
- Lubrication: the product has designed for a minimal maintenance; however, check every six months the lubrication on the moving & mechanical parts, this would ensure an optimal duration.

ASSEMBLING:

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May be differences from reality.

The product is assembled and ready for use.

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INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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0805401:

- 1) Roll out the main section of the ramp so it is straight.
- 2) Line up the side section of ramp ensuring the cut out on this section is at the bottom of the ramp. (the top of the ramp is where two holes are drilled through).
- 3) Lock the sections with the main frame together using the latch on the underside of the ramp.
- 4) Repeat points 2 and 3 for the other side section.
- 5) To fold : Release both latches underneath the ramp.
- 6) Remove both side sections.
You can now roll up the main ramp frame.

0805402 & 0805403:

- 1) Carry the ramp to the required area using the carry handle.
- 2) UnFold the ramp.
- 3) Position the ramp so the top of the ramp is higher (the top of the ramp is where two holes are drilled through).
- 4) The top of the ramp has a lip to allow secure and comfortable operation.
- 5) Ensure this lip is fully on the step or car entrance and at 90° to the object

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WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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0805400:

- 1) To extend the ramp to it's full length, push the button and slowly pull out the narrowest section of the ramp while holding the widest section, until it is locked in position.
- 2) Repeat this for the other ramp section.
- 3) This ramp CANNOT be permanently installed.

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