GR EN

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INTENDED USE

The bath chair is designed to allow disabled people to be able to enjoy a bath while sitting on it. Can be used inside or outside of the tub and the shower, reducing muscle fatigue due to standing position as well as slip and injuries risk.

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ASSEMBLING:

The product is, assembled ready to use. You may need to assemble part of the back or the metal legs.

SYMBOLS

Please refer on **ANNEX A**

MAINTENANCE & CLEANING

Clean every part of the bath chair often. Use only a damp cloth and mild soap on surfaces that need cleaning. Then dry well before using the product. If you must disinfect the product, use a common, mild disinfectant.

IMPORTANT!

Before using it take care of the following:

- That the chair is correctly assembled
- That the legs end firmly touching the floor or on the bathtub.

WARNING

If trouble occurs in any of the above, **DO NOT** use the seat until the problem is resolved.

FURTHER INSTRUCTIONS

- Position the on the floor and check that it is correctly in this position.
- If necessary, adjust the height.
- Transfer the user in the chair.
- Make sure that the chair is secure and does not swing before starting the bath.
- Do not leave the user alone, unsupervised.

ENVIRONMENTAL PROTECTION

If one day you find that your device needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your appliance along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your appliances must be disposed.
- 3) Correct disposal of your appliance helps the protection of the environment as well as the recycling of the appliance components.

E.C. DECLARATION OF CONFORMITY

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can be worn out during use or over time or from water, like the upholstery, wheels, brakes, wood, plastic parts, or metal parts. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and carriers are not covered by the warranty. No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
DDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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