**GR EN**

**MOBIAKARE**<sup>®</sup>  
M E D I C A L E Q U I P M E N T

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## **INTENDED USE:**

These wheelchairs have been designed for physically or mentally handicapped people with limited mobility. It makes possible to transport people with mobility problems or limited mobility and can be used for the toilet and personal hygiene with the help of an assistant.

These wheelchairs can be used as a toilet seat.

## **SPECIFICATIONS**

Please refer on **ANNEX A**

## **DIMENSIONS**

Please refer on **ANNEX B**

## **ASSEMBLY**

Please refer on **ANNEX A**

## **SYMBOLS**

Please refer on **ANNEX B**

## **GENERAL WARNINGS:**

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.
- Getting in and out of the wheelchair may be performed independently, provided you have an adequate mobility and upper body strength.

## **IMPORTANT!**

The construction of this wheelchair is designed to provide maximum safety. However, users may be at risk by using their wheelchairs unsuitable. For your own safety, the following rules must be observed. Nonspecific or incorrect adjustment work can increase the risk of accidents.

- The Commode wheelchairs 0808378, 0806777, 0808396, 0806053 should only be used indoors.
- Be careful during your first course in this wheelchair.
- Meet your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads or motorways.
- Do not attempt to climb or descend with water, ice, or oil membrane.
- Do not use unauthorized parts, accessories, or adapters other than those approved by MOBIAK.
- Do not stand in the wheelchair.

## **BEFORE USE:**

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check, if the product is properly assembled.
3. Check if the seat is completely open. If not push the seat down from the lateral frame.
4. Check that the brake works correctly.
5. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
6. If there are quick release axes, check that they have been secured correctly.
7. Check tires, tire pressures (if need id) and wheel locks (brakes). The brakes will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.

## **WARNING!**

The following methods must be used only from experienced patients. If you are not experienced, ask for help from an attendant.

## **GETTING IN THE WHEELCHAIR:**

1. Position the wheelchair as close as possible to you.
2. Engage wheel locks.
3. Hold on to your assistant and slowly sit in position.

## **GETTING OUT OF THE WHEELCHAIR:**

1. Position the wheelchair as close as possible alongside the seat to which you are transferring.
2. Engage wheel locks.
3. Get support from your assistant.
4. Shift body weight into the seat that you are going to transfer.

## **OTHER INSTRUCTIONS:**

1. Avoid stairs or obstacles, otherwise ask help from your attendant:
2. This wheelchair must be used solely to carry a person at a time on the seat.
3. Users should not sit in wheelchair while are using any means of transport.
4. The wheelchairs have a brake on the back wheels, which should be secured when the wheelchair is not moving.
5. DO NOT push on the footrests to sit or stand up, DANGER OF OVERTHROW. The footrests are used by the user when He or She is seated.
6. DO NOT touch moving parts while moving DANGER OF INJURY

## **MAINTENANCE:**

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

### **WEEKLY**

1. Check the commode status for any breaks, cracks, or leaks.
2. Check the steel frame screws. In case you find loose please tighten it. Clean the fabric with soapy water or sufficient products.
3. Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
4. Check the wheel brakes and confirm their function.
5. Check for damage to front and rear wheels.

### **QUARTERLY**

1. Grease or oil all components that are subject to friction or impending wear.
2. Tighten the screws and nuts.
3. Check stability (without wheels) or balance (with wheels) of the toilet
4. General revision of the chair's status and repair to whatever is necessary.

## **ANNUALLY**

1. We recommend a full review once a year to a qualified service provider.
2. For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

## **CLEANING:**

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- **DO NOT leave the wheelchair wet.**
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. it is dangerous use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

## **CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

## **ENVIRONMENTAL PROTECTION**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

## **DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

## **INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

**WARRANTY:**

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels or brakes.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, supply problems (where applicable), carriers are not covered by the warranty

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

**WARRANTY CLAIM FORM**

<b>BUYERS DETAILS:</b>			
<b>FULL NAME:</b>			
<b>PURCHASING DATE:</b>		<b>PHONE::</b>	
<b>SERIAL NUMBER:</b>			
<b>LOT :</b>			
<b>DEALER DETAILS</b>			
<b>FULL NAME:</b>			
<b>DDRESS:</b>			
<b>PHONE:</b>			
			<b>SIGN AND SIGNATURE</b>

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The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar. The wheelchair comes part assembled and we need to add the backrest, armrest, and footrest.

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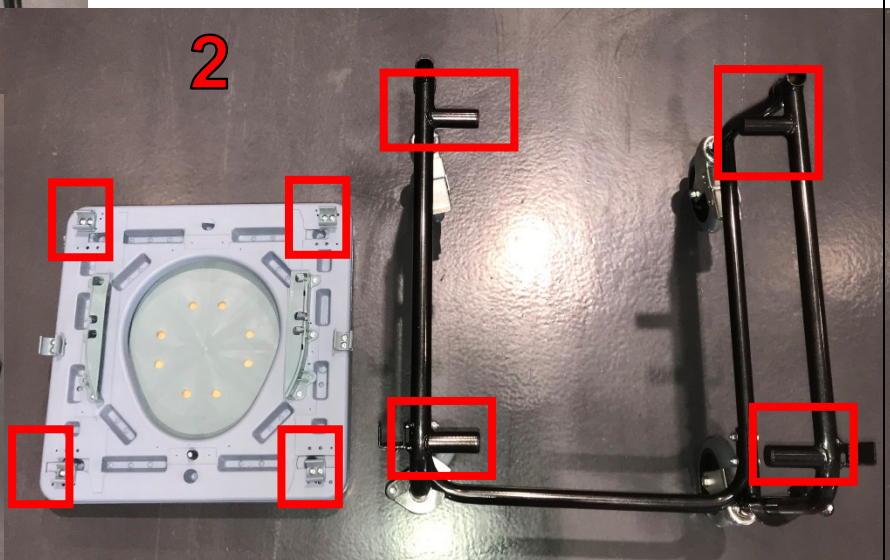
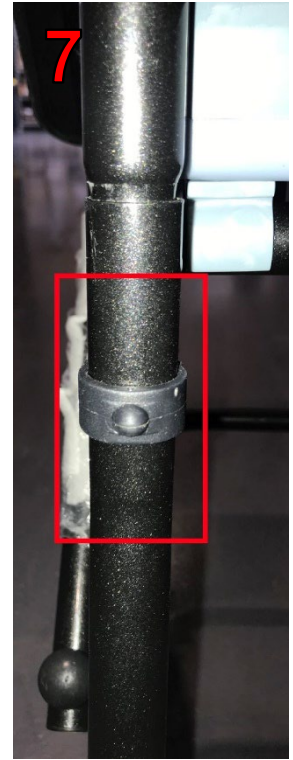
*Remove the "C" clips, insert the back into the corresponding slots and secure with the same "C" clips.*

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
*Secure the armrests into the corresponding locks.*  
*Like the pictures 8 & 9*

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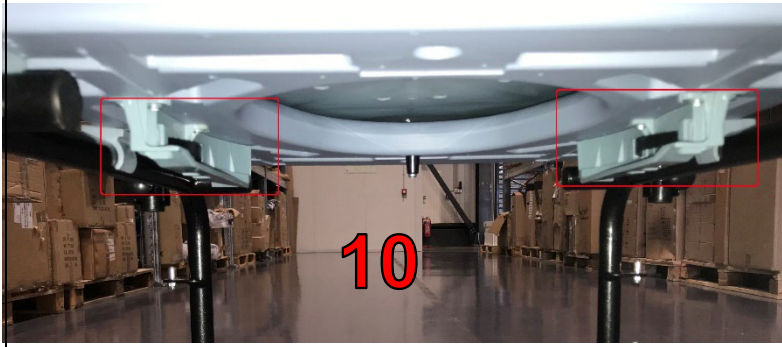***To raise the folding armrest, pull the lever out or press the safety button and pull upwards or downwards.***

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*Place the plastic commode into the corresponding locks.  
Like the pictures 10 & 11*

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*Assembly legrest by installing the hinge plates that are on the part onto the hinge pins on the wheelchair frame. Push the legrest towards inside of the wheelchair until it locks into place. Repeat the same procedure for the other legrest.*

*Remove the legrest by pushing the release lever, rotate outwards, lift the legrest off the hinge pins.*

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*The two rear steering wheels have brakes. Setting brakes: press brake lever down with your foot. Releasing brakes: pull the lever upwards with your toe.*

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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