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T: +30 28210 63222 | WEB: www.mobiakcare.com

INTENDED USE:

The Beds 0808470 & 0808471 is used for patients who have health problems, and they need to stay lying in bed.

They are designed for people who need to be treated for health reasons or disability in nursing homes, hospitals or even in healthcare facilities.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- **NEVER PERMIT ANYONE TO BE UNDER THE BED OR IN BETWEEN THE RAISED BED FRAME COMPONENTS AT ANY TIME.**

GENERAL WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury. Keep all moving parts and all drive shafts free of obstructions during operation of the bed.
- Do not attempt to lift the product by any removable parts
- Never plug into an outlet that is greater than 220 Volts AC.
- Never operate the bed if it has a damaged cord or plug. If the bed is not working properly call qualified technician.
- Keep all cords away from heated or hot surfaces. i.e. radiators, heating pads, portable gas heaters.
- Do not open parts such as the motor, hand control unit, or electrical control box.
There are no user serviceable parts inside these components. Only trained service personnel are qualified to repair these parts. If uncertified individuals perform any work on these beds, the warranty cannot be applied.
- When the bed is not going to be used for an extended period, UNPLUG electric bed from the wall outlet.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

PRECAUTIONS FOR USE:

- Do not exceed the maximum load capacity for any reason.
- When lowering the head or foot section, make sure that nobody inserts hands, legs or other body parts under the bed base
- Make sure that the floor surface supporting the bed is flat.

BEFORE USE:

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
2. In case of damage do not use the product and contact your dealer for further instructions.
3. Check, if the product is properly assembled & operate correctly and all screws are secure and well screwed on.
4. Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects.
5. Ensure that the head section is secured with the head-board and foot section is secured with foot-board.
6. Ensure that the hand control cable and other power cables are routed and secured properly so these cables will not become entangled, pinched and/or severed during operation of the electric bed. NEVER OPERATE THE BED IF THESE CORDS ARE DAMAGED.
7. To prevent casters from moving whenever the bed is stationary, ensure the bed caster locks are properly engaged.
8. ALWAYS lock the caster brake when transferring into or out of the bed. Inspect the caster brake BEFORE using the bed to ensure the brake function adequately.
9. Never permit more than one person on/in the bed at any time.
10. Do not lay, sit or lean in such a way that the entire body weight is placed only on RAISED head or foot sections of the bed. This includes when repositioning or transferring in or out of bed.
11. Keep the bed a minimum of 35-40 cm away from any direct heat source or wall.

MATTRESS SELECTION:

Use mattresses or anti-decubitus mattresses according to the prescription of the treating Physician. The size of the mattress should not be larger than the frame of the bed. All MOBIK mattresses and anti-decubitus are suitable for beds.

Ensure the mattress is centered on the frame. This will minimize the risk of the user becoming trapped between the mattress and any bed rail or raising parts.

		0808470	0808471
1	FULL LENGTH	224 cm	224 cm
2	FULL WIDE	96 cm	96 cm
3	LYING SURGASE LENGTH	202 cm	202 cm
4	LYING SURGASE WIDE	88 cm	88 cm
5	HEIGHT MIN	40 cm	40 cm
6	HEIGT MAX	58 cm	58 cm
7	LENGTH FOR BACK PART	70 cm	70 cm
8	LENGTH FOR MIDDLE PART	31 cm	31 cm
9	LENGTH FOR LEG PART	70 cm	70 cm
10	BACK TILTING	70°	70°
11	KNEE TILTING	45°	45°
12	QTY OF MOTORS	2 PCS	1 PC
13	MOTOR 9V BATTERY	YES 1 PC	YES 1 PC
14	HAND CONTROL	YES	YES
15	METAL CRANK	NO	1 PC, FOR HEIGHT RAISING
16	TRENDELENBURG - ANTITRENDELENBURG	NO	NO
17	MAX LOAD	200 Kg	200 Kg
18	MAX PATIEN WEIGHT	150 Kg	150 Kg
19	OVERALL BED MAXIMUM WEIGHT CAPACITY	200 Kg	200 Kg
20	NET WEIGHT	80 Kg	75 Kg
21	GROSS WEIGHT	86 Kg	81 Kg
22	FRAME	STEEL	STEEL
23	COLOR	BROWN	BROWN
24	PAINT	NORMAL TREATMENT, DURABLE POWDER COATING	NORMAL TREATMENT, DURABLE POWDER COATING
25	WHEELS	4 CASTORS, 2 WITH BRAKE	4 CASTORS, 2 WITH BRAKE
26	HEAD-FOOT BOARD	METAL WITH WOODEN PANEL STICKER	METAL WITH WOODEN PANEL STICKER
27	MONKEY POLE BASE	STANDARD	STANDARD
28	MONKEY POLE	OPTIONAL (CODE 0809251)	OPTIONAL (CODE 0809251)
29	MATTRESS HOLDER	YES	YES
30	MOTOR 9V BATTERY	YES	YES
31	CARTON DIMENSION HEAD	94x22x111cm	94x22x111cm
32	CARTON DIMENSION FOOT	94x22x114cm	94x22x114cm
33	CARTON DIMENSION RAILS	129x6x49.5cm	129x6x49.5cm

[1] Footboard

[2] Leg Rest

[3] Thigh Rest

[4] Backrest

[5] Headboard

[6] Lifting pole brackets

[7] Controller

[8] Rails

[9] Motor for Back-rest and Leg-Rest raising.

[10] Castors

[11] For 0808471, mechanical Crank, height raising.

[12] Mattress and Pillow is not included

ASSEMBLING:

For 0808471: Crank Handle

Foot Board

Head Board

Foot Section


Head Section

For 0808470 only

Bracket for Hi/Low Motor

4 Casters

Drive Shaft

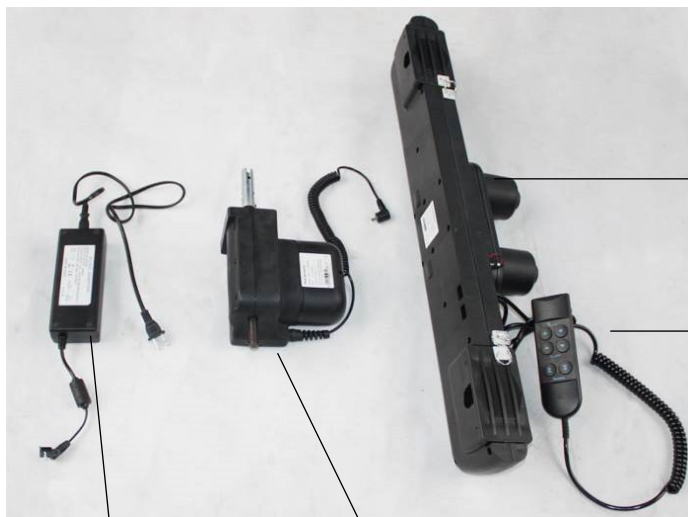Motor Box for Full-Electric and Semi-Electric Bed

Hand Control for Full-Electric or Semi-Electric Bed

For 0808470 Only
Adaptor

For 0808470 Only
Hi-Low Motor

The Product will arrive in three (3) cartons with the following contents:

1 carton with the Head section, 1 carton with the Foot section, 1 carton with the rails.

NOTE: The illustrations for the assembly are indicative to make the assembly easier.
May will be differences from reality.

CONNECTING THE TWO FRAMES:

Figure 1

Figure 2

Figure 3

7. Place the Head Section on its side to your left with the center mounting hooks to the right. (See figure 1).
8. Place the Foot Section on its side to your right with the center mounting rivets facing to the left.
9. The head and foot sections should now be on their sides at approximately a 90° angle to one another.
10. Hook the head section center mounting hooks to the foot section center mounting rivets. (See figures 2 and 3)
11. After head and foot sections are connected, slowly open the frame sections into a horizontal (straight) position while still on their sides. Make sure the hooks remain engaged on the rivets at all times. (See figures 4)
12. After you join them, place the screw & nut.

Figure 4

TO INSTALL CASTERS:

- 1) Position the two locking casters so they will be diagonally opposite from one another when the bed is assembled.
- 2) Insert the shaft of the caster into the caster socket of each bed leg. Ensure the stem is fully inserted into the socket.

- 3) To prevent excess movement of the bed during assembly, lock each of the two locking casters by pushing down on the caster lock.

Note: Locking the casters may not prevent the bed from moving on slick or slippery surfaces.


- 4) To unlock the two locking casters, pull up on the caster lock.

INSTALLING THE HEADBOARD AND FOOTBOARD:

CAUTION: Do not place your hand between the rectangular tube section and the bed ends while attaching the bed boards as injury may occur.

To Attach the Head and Foot Boards:

1. Stand the head board section as close to the head frame as possible.
2. Raise the head frame until the rivets on the corner plates of the head frame fit into the corner locks on the headboard. The headboard may need to be tilted slightly forward for the rivets to slide into the locks.
3. Return the headboard to its full upright position to lock it into place.
4. Repeat steps 1-3 for the "Foot Board" section.

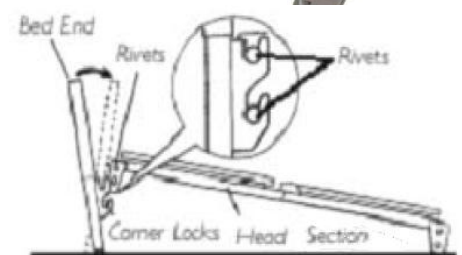BED OPERATION

Hand Control: Semi-Electric 0808471 & Full-Electric Beds 0808470:

Semi-Electric Hand Control

Up & Down button for Backrest positioning.

Up & Down button for Footrest positioning.

Up & Down button for Bed Height positioning.

Full-Electric Hand Control

For the operation of the Hand controller, after connecting the product to a 220 Volt power supply, press the "up arrow" button to adjust the moving up and the "down arrow" to adjust the moving down.

Battery Backup:

In the event of the power failure, the standard 9 volt battery in the motor, will allow the user to lower the head and foot sections of the bed. Will NOT allow the bed's motor to lift the head or foot sections.

NOTE: Periodically replace the battery.

NOTE: The Battery is not included in the packaging.

Installing the main Electric Motor Unit:

1. Before installing:
 - A. **Semi-Electric Bed 0808471** - make sure that the hand control cable is plugged into the motor.
 - B. **Full-Electric Bed 0808470** - make sure that the hand control and the Hi/Low Motor cables are plugged into the motor.

Hi/Low Motor Cable

Hand Control Cable

2. Connect the 9V battery (sold separately) to the battery clip on the motor. It is stored in the recess on the side of the motor. With this battery installed, the motor can be operated to return the sleep surface to the horizontal position in the event of a power failure.

3. Locate the slide caps. Remove the slide caps from both ends of the motor.

4. Prior to installation, make sure that the 2 motor blocks are in the retracted position.
5. **For Full-Electric Bed Only** - Insert the adapter into the clip.

Aligning and Mounting the Motor:


5. With the motor oriented properly underneath the bed (head to head, foot to foot), align the motor with actuator bar and lift the head section to the highest position. The motor will automatically be pulled towards the bed and lock onto the actuator.
6. Once the head side of the motor is attached to the head section of the bed, return the head section to the flat position.
7. Once flat, attach slide caps to the head section of the motor and slide it into place. Repeat for foot section.
8. Completed Assembly.

Installing the Hi/Low Motor on the Full-Electric Bed:

Make sure the slot on HI/Low Motor engages the roll pin on the gearbox axle.

1. Connect the drive shaft assembly to the bed by first attaching either side of the drive shaft to the head board gearbox.
2. **For the Semi-Electric Bed (0808471)** - attach the other end to the foot board gearbox.
3. **For the Full-Electric Bed (0808470)** - attach the other end to the foot board Hi/Low Motor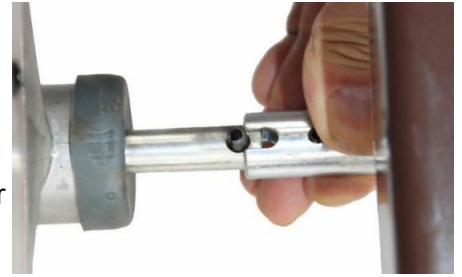

**Head Board Gearbox Box
For Attachment to Both
Semi-Electric and Full-Electric Bed**

**Foot Board Gearbox Box for
Semi-Electric Bed Only**

**Hi/Low Motor for Full-
Electric Bed Only**

HAND CRANK FOR THE SEMI-ELECTRIC BED:

The manual bed crank is located at the foot of the bed.

3. To raise the bed - Turn the crank handle clockwise.
4. To lower the bed - Turn the crank handle counter-clockwise.

If using the semi-electric bed footboard with a full-electric bed/motor, first remove the crank handle from the footboard. If left in place, the crank will turn when the motor is on and could cause personal injury or damage to the bed.

DISASSEMBLY:

Follow the steps described above in reverse order. When disassembling the bed, make sure all parts are in the lowest position.

MAINTENANCE, CLEANING AND DISINFECTION:

Unplug the bed before cleaning. When using liquids on or around the bed, you should be careful not to get wet the electrical parts.

If a liquid is spilled in or around the electric bed, clean up the spill and let the bed or area around it dry thoroughly before using the electric controls again.

Use only a wet cloth and mild soap on surfaces where they need cleaning. Then dry thoroughly before you use the product.

DO NOT leave the product wet. allow them to dry completely before using the bed.

Periodically raise the head and foot sections of the bed and remove any dust from the frame.

Periodically remove the mattress and clean the mattress deck.

Do not use chemical cleaners to clean the frame. This could cause damage to the surface.

If you need to disinfect the product, use a common, mild disinfectant.

Protect the item from scratches, cuts and punctures.

Periodically Check all electrical & mechanical bed components for damage or excessive wear (i.e. cracked, frayed or broken housings, or worn components).

Periodically Check all bed functions: Head-rest raises and lowers properly. Foot-rest raises and lowers properly. Bed height raises and lowers properly.

Keep electrical parts away from dust & moisture.

Lubricating. Lightly grease all actuator, screw, threads with white lithium grease. Lubricate all caster roller and swivel bearings with light machine oil.

Routine cleaning and electrical inspection should be performed by the user. All repair and maintenance on the bed should be performed by a qualified technician. After any adjustments, repair, or service and BEFORE use, make sure all attaching hardware is tightened securely

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, brakes, wood, or plastic parts.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable), carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

BUYERS DETAILS:			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SERIAL NUMBER:			
LOT :			
DEALER DETAILS			
FULL NAME:			
DDRESS:			
PHONE:			
			SIGN AND SIGNATURE

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TRENDELENBURG & REVERSE TRENDELENBURG POSITION, HOW YOU CAN DO IT:

In the Trendelenburg position, the body is lain supine, or flat on the back on a 15–30 degrees incline with the feet elevated above the head.

The reverse Trendelenburg position, similarly, places the body supine on an incline of 15–30 degrees but with the head now being elevated.

The Trendelenburg position is used in surgery, especially of the abdomen and genitourinary system. It allows better access to the pelvic organs as gravity pulls the intra-abdominal organs away from the pelvis.

The position was named for the German surgeon Friedrich Trendelenburg (1844–1924).

THE TRENDELENBURG FUNCTION MAY BE USED ONLY AFTER DOCTORS PRESCRIPTION & EXCLUSIVELY UNDER THE SUPERVISION OF MEDICAL PROFESSIONALS.

Trendelenburg :

1. *Lower the bed to its lowest height. Disengage the drive shaft by compressing the spring fitting and removing the drive shaft from the Hi/Low Motor to the head board gearbox.*
2. *Raise the foot end of the bed using the hand control or crank. The head end of the bed will remain in the low position.*
3. *When the Trendelenburg position is not used, lower the foot end of the bed to its lowest height using the hand control or crank, install the drive shaft back to the Hi/Low Motor.*

Reverse Trendelenburg :

1. *Raise the bed to its highest height. Disengage the Hi/Lo drive shaft by compressing the spring fitting and removing the drive shaft from the motor to the foot end of the bed.*
2. *Lower the end of the bed using the hand pendant or crank. The head end of the bed will remain in the high position.*
3. *When the Reverse Trendelenburg position is not used, raised the foot end of the bed to its highest height using the hand control or crank, install the drive shaft back to the Hi/Low Motor.*

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