**GR EN**

**MOBIACARE<sup>®</sup>**  
M E D I C A L E Q U I P M E N T

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**MOBIAK**<sup>®</sup>  
MEDICAL • GASES • FIREFIGHTING est. 1977

## INTENDED USE

The Commode Chair can be used by people with limited mobility who are unable to go to the toilet in the house and as an aid to the toilet bowl without the container.

Designed for indoor use only.

**0809155**  
**Foldable**

**0809156**  
**Foldable –**  
**with Castors**

## SPECIFICATIONS

- Frame made of steel.
- Adjustable height.
- With castors (Ref: 0809156)

## DIMENSIONS

REF.:						
<b>0809155</b>	45cm	41cm	54cm	46-56cm	7Kg	100Kgr
<b>0809156</b>	45cm	41cm	54cm	45-56cm	7,2Kg	100Kgr

## SYMBOLS

Please refer on **ANNEX A**

## ASSEMBLY

Please refer on **ANNEX B**

## MAINTENANCE & CLEANING

Clean the toilet container immediately after use. Initially, rinse with warm water. Then use a soft cloth impregnated in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.

**ATTENTION:** Do not use acidic & caustic dilutions.

We recommend that you clean the product with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.

Do not smoke & do not use any flame near the product.

Protect the product from scratches, cuts and punctures.

## IMPORTANT!

Before using the Commode Chair make sure that:

- Is in full extension and is locked in this position.
- The wheels (if available) are firmly fixed.
- The seat is correctly placed and secure in this position.
- The tank is correctly placed into the appropriate slot.
- There are no cracks or fractures in the frame.

## WARNING

If trouble occurs in any of the above, **DO NOT** use the commode chair until the problem is resolved.

### FURTHER INSTRUCTIONS

- When sitting or standing up from the commode chair push the handles down, securing that it will stand still and not slip.
- When you get out of the commode must rely on both handles equally. Leaning only to one handle (without escort) can cause injury or damage the product.
- Regularly clean the commode chair.
- **Do not let the product wet.**
- Do not let children near the commode chair without supervision.

## EC DECLARATION OF CONFORMITY

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

## INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides

## ENVIRONMENTAL PROTECTION

If one day you find that your device needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your appliance along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your appliances must be disposed.
- 3) Correct disposal of your appliance helps the protection of the environment as well as the recycling of the appliance components.

## WARRANTY

The product has a guarantee of good operation 1 year. The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product instructions for use. It also does NOT cover worn parts such as brakes, plastic parts, and whatever it is going to be damaged during use or over time or from water. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), transport companies are also not covered by the warranty.

## WARRANTY CLAIM FORM

<b>BUYERS DETAILS:</b>			
<b>FULL NAME:</b>			
<b>PURCHASING DATE:</b>		<b>PHONE::</b>	
<b>SERIAL NUMBER:</b>			
<b>LOT :</b>			
<b>DEALER DETAILS</b>			
<b>FULL NAME:</b>			
<b>DDRESS:</b>			
<b>PHONE:</b>			
<b>SIGN AND SIGNATURE</b>			

MOBIAC S.a  
 Kathiana Akrotiriou  
 Chania-Crete  
 +30-28210-63222 [www.mobiakcare.com](http://www.mobiakcare.com)

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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**NOTE:** The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality. Assembly tools are included in the package.

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Lift the connecting side parallel to the back. Then place the curves in the corresponding positions and we press until it will locks.

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Place the container in the correct position (from the front side of the wheelchair).

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