PL

WYPOSAŻENIE

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EN

COMPOSITION

- PULMOGAIN* is composed by:
- 1 transparent central body divided into three chambers made of copolymer and non-toxic polystyrene showing 600, 900 and 1200 cc per second. The value of each chamber expresses the volume exhaled per second.
 - 1 base made of non-toxic polypropylene.
 - 3 spheres made of non-toxic polypropylene.
 - 1 corrugated tube with mouthpiece made of polyethylene.
 - 1 air filter.

INSTRUCTIONS OF USE

Place *PULMOGAIN* at mouth level, exhale normally before inhaling.

- 1- Close your lips around the mouthpiece.
- 2- Inhale thoroughly until lifting the three balls (if possible).
- 3- Hold inspiration as long as possible (apnea).
- 4- Remove the mouthpiece from your mouth and exhale.
- 5- Repeat this exercise.

INDICATIONS

To prevent post-operative complications of the respiratory tract, e.g. atelectasis, pneumonia or pulmonary disorders.

CONTRAINDICATIONS

For any contraindications, consult the doctor or therapist.

HYGIENE

PULMOGAIN can be fully disassembled and each part shall be washed and rinsed with water and soap. The air filter shall be replaced every 10 to 15 days of use. *PULMOGAIN* is a single-patient device and must be used during the period prescribed by the doctor. Wash the mouth piece with running hot water (about 40°C) after use and leave it to dry. It is possible to position the pipe with the mouth piece in its seat on the right of the body, mouth piece facing down. When not in use, keep the device in a dry, clean place. The device does not contain latex and can be disposed of in the differentiated collection plastic container.

WARNING

Following the instructions of your doctor, nurse or therapist while using *PULMOGAIN* is important. Store in dry place at room temperature. In the event of slight dizziness during the procedure, stop and rest before continuing. In paediatric use, children must be supervised during use. As for any medical device, it must be kept out of the reach of children. The data flow stated in every cylinder is given as an indication for the doctor or nurse. The filter is found in the point of insertion for the pipe and is used only to prevent the inhalation of particles.

Notice to the user and/or patient: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Product Ref.:

RE400100 individually boxed
RE400110 in nylon bag

Spare accessories:

Air Filter (bag of 10's): ref. SP0154
Mouthpiece: ref. SP0155
Corrugated Tube: ref. SP0156

Pulmo-gain

CA-MI srl - Via Ugo La Malfa 13
Frazione Pilastro - 43010 Langhirano (PR) - Italia
Tel. +39 0521 637133 - 631138 Fax +39 0521 639041
vendite@ca-mi.it - export@ca-mi.it www.ca-mi.it

Pulmo-gain

PULMOGAIN is a Single-Patient Respiratory Exerciser to train respiration through deep inspiration. Deep breathing helps to expand the pulmonary alveolus, as well as clean the passage from mucus in the respiratory tract.

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