GR EN

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INTENDED USE

The lift chair with wheels is intended for disabled, sick, and elderly people as well as people with mobility difficulties.

It can be used to transfer the patient to and from the bed/chair and/or for use by people who are unable to go to the toilet due to limited mobility.

SPECIFICATIONS & DIMENSIONS

CODE 0811900

FRAME	STEEL
SEAT MATERIAL	ABS
SEAT CUSHION	REMOVABLE WITH WASHABLE COVER
BACKREST AND ARMPADS MATERIAL	PE
REAR CASTORS	3"
FRONT CASTORS	5"
BRAKES	IN FRONT AND REAR CASTORS
FOOTREST	FOOTREST PLATFORM

The lift chair with wheels is suitable for indoor use and the transport speed is ≤ 3 km/h.

0811900	46cm	40cm	60 cm	71 cm	79-99 cm	37-57cm	3"5"	100kg	25.5kg

CODE 0811950

FRAME	STEEL
SEAT MATERIAL	ABS
SEAT CUSHION	REMOVABLE WITH WASHABLE COVER
BACKREST AND ARMPADS MATERIAL	PE
REAR CASTORS	3"
FRONT CASTORS	5"
BRAKES	IN FRONT AND REAR CASTORS
FOOTREST	FOOTREST PLATFORM
WITH HYDRAULIC LIFTING SYSTEM	THE HYDRAULIC PUMP CONTROLS THE LIFT OF THE SEAT BASE AND THE LEG LIFT SYSTEM.

The lift chair with wheels is suitable for indoor use and the transport speed is $\leq 3\text{km/h}$.

0811950	46cm	40cm	59 cm	77 cm	93-119cm	44-70cm	3"5"	120kg	32kg
----------------	------	------	-------	-------	----------	---------	------	-------	------

SYMBOLS

Please refer on ANNEX A

ASSEMBLY

Please refer on ANNEX B

DETAILED INSTRUCTIONS FOR USE IN A PATIENT

Please refer on ANNEX C

MAINTENANCE

The purpose of periodic check of the lift chair with wheels, is to confirm its proper operation as an aid and must be done at least once a month.

- Check the container status for any breaks, cracks, or leaks.
- Check the frame screws. If any are found to be loose, please tighten them.
- Check the plastic parts and frame of the lift chair with wheels for any looseness, cracks or breaks.
- Check the stability of the lift chair with wheels
- Check the wheel brakes and confirm their operation.
- Check the condition of the front and rear wheels.

In case of repair, use only authentic spare parts.

CLEANING

- The user or the attendant of the wheelchair is responsible for the cleaning, inspection and maintenance.
- In addition to maintenance and inspection, a general cleaning for the frame and the upholstery can be carried out.
- DO NOT leave the product wet or dirty with dust or mud.
- The frame should be cleaned regularly. We recommend cleaning the frame and seat with a mild detergent or antiseptic solution. For your own convenience you can use a spray bottle after adding the solution. Spray the product and then clean with a sponge. Wipe the surfaces dry with a clean cloth.
- Use a clean and soft cloth to wipe the surface of the product.
- The commode must be cleaned immediately after use. First, rinse with plenty of warm water. Then use a soft cloth soaked in a mild detergent or antiseptic solution. Wipe with a clean, soft cloth.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.
- CAUTION: Do not use acidic or caustic solutions.
- The pillow cover is removable and hand-washable in cold water.

IMPORTANT(!)

Before using the lift chair with wheels **MAKE SURE** that:

- It is fully extended & locked in this position.
- The wheels are well secured.
- The seat fits correctly & is locked in place.
- The container is correctly placed in the appropriate slot.
- There are no cracks or breaks in the frame.
- Do not smoke & do not use any flame near the product.
- Protect the product from abrasions, cuts and punctures.
- Before each use, make sure the seat and back are securely fastened.
- Adjust it to the proper height before use and make sure the seat panels on both sides remain at the same height.

- When the user sits on or gets up from the lift chair with wheels, they should not stand on the footrest as there is a high risk of tipping over. The user must during this process stand on the ground.
- The caregiver-attendant must not stand on the footrest as there is a high risk of tipping over and damage of product.
- When using the lift chair with wheels, make sure the back safety lock and seat belt are locked.
- When the user sits in or gets up from the lift chair with wheels, make sure the wheels are locked.
- When adjusting the seat height, the front wheels must be locked and the handles on both sides must be rotated at a constant speed. **(refers to code 0811900)**
- When adjusting the seat height, the front wheels must be locked and the caregiver controls the height by pressing the pump foot lever. **(refers to code 0811950)**
- The lift chair with wheels should only be used indoors on a flat, non-slip surface to ensure stability.
- In case of use on sloping ground, all four wheels must be locked.
- Use should always be under the supervision of a family member, nurse or attendant.
- Increase caution when using the product on carpets, wet or slippery surfaces.
- Do not use on icy surfaces.
- Never exceed the maximum user weight.
- **For the 0811950, on the lifting mechanism, if there is water or some other fluids from the patient, clean and dry it immediately.**

OTHER INSTRUCTIONS

- When sitting or standing up, you must press down on the handles so that the lift chair with wheels remains stationary and does not slide.
- When standing up you must lean on both handles equally. Support with only one hand (and without an attendant) may cause damage or injury.
- Clean the lift chair with wheels regularly.
- Do not leave the lift chair with wheels wet.
- Do not leave children near the lift chair with wheels without supervision

SPECIAL STORAGE CONDITIONS AND METHODS

The packaged product should be stored in a well-ventilated and clean room with a temperature not exceeding 0 °C ~ 40 °C, relative humidity not exceeding 80%, no corrosive gases and good ventilation.

During transport, it should be protected from sources of moisture, light and heat.

WARNING

If a problem occurs with any of the above, **DO NOT** use the chair until the problem is corrected.

- Do not use the product for purposes not included in this manual.
- The manufacturer disclaims all responsibility for any consequences arising from improper use of the product.
- The manufacturer reserves the right to change the instructions contained herein without notice.
- Do not allow children to play with the lift chair with wheels
- Do not attempt to lift the lift chair with wheels from any assembly part.
- Keep the lift chair with wheels away from sources of heat.

- Getting into and out of the lift chair with wheels without assistance requires adequate upper body strength and mobility.
- Do not use the lift chair with wheels on stairs (simple stairs or escalators).
- Should not be used in the shower. Its frame and wheels could rust.

ENVIRONMENTAL PROTECTION

Should you find one day that your product needs to be replaced or if it is of no further use to you, think of the protection of the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your products must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product components.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

WARRANTY

The lift chair with wheels is guaranteed for 2 years from the purchase date. The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product instructions for use. It also does NOT cover worn parts during use. Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where provided), transport companies are also not covered by the warranty. Shipping costs for goods of warranty are to be paid by the customer or by dealer.

WARRANTY CLAIM FORM

BUYERS DETAILS			
FULL NAME:			
PURCHASING DATE:		PHONE:	
SELLER DETAILS			
FULL NAME		ADDRESS	
PHONE		SIGN AND SIGNATURE	

E.C. Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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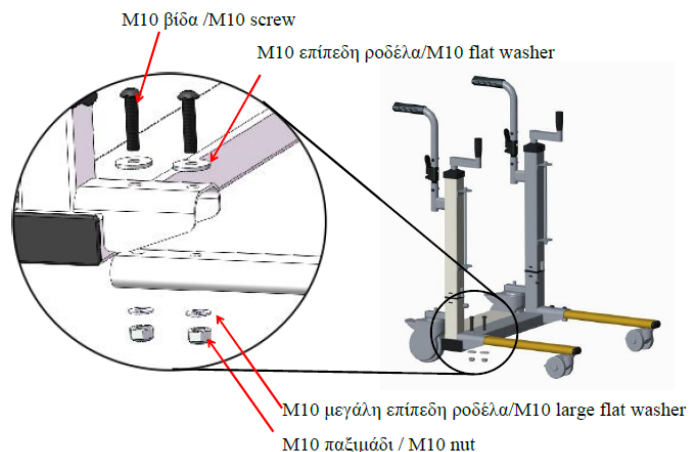
Place the tubes of the small wheels into the corresponding tube slots of the base (left and right) and fix them with total 4 M10 screws, 4 large M10 flat washers, 4 M10 flat washers (smaller in size) and 4 M10 lock nuts as shown in the picture. On each side use two pieces of each component.

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
Place the left seat frame in the center of the two "projections" of the left column tube of the stand, pass the pin through the column, install it, and fix it with the M8 nut (as shown in the picture). The right seat frame is placed in the same way.

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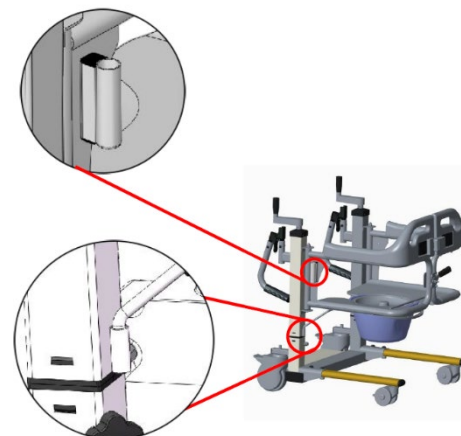If you need to use the commode pot, first install the iron frame of the toilet into the fixing holes of the two pipes of the base, and then install the commode pot from the back as shown in the picture.

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After all the parts are installed, lift the handle and then you can use it normally.

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
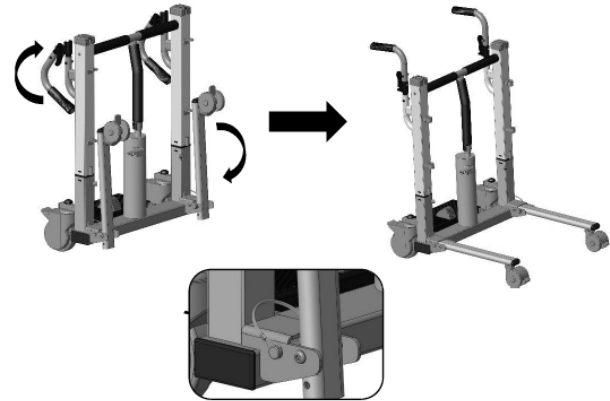
Pull the handle up to the use position, pull out the two $\varnothing 8$ D-shaped pins on the base, turn the support legs down to a suitable position, pass the D-shaped pins back to the original position and fix the support legs.

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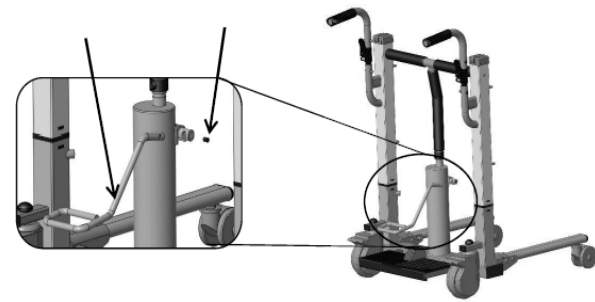
Install the hydraulic pump foot control into the appropriate position of the hydraulic pump hole and use the M8 headless screw that has been locked on the hydraulic pump to lock and fix it without shaking.

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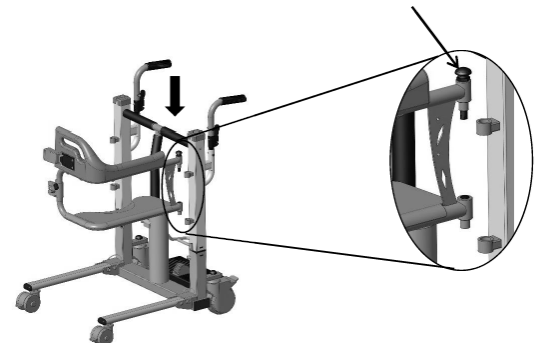
Place the right seat base on the right column tube of the base at the position shown in the figure. After pressing the quick release shaft cap, align the holes and install it. After installation and debugging, the seat base is not obviously loose and can rotate flexibly (use the same method to install the left seat base)

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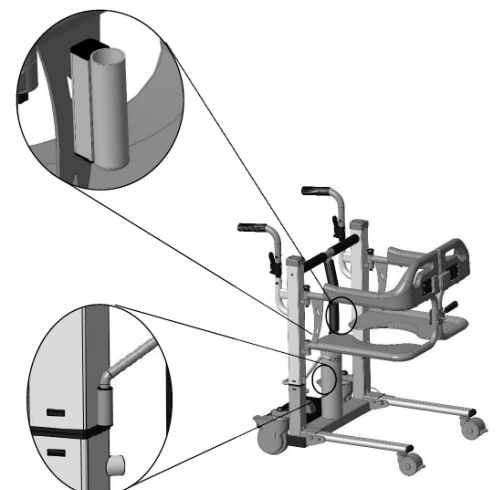0811900 & 0811950

ANNEX C. DETAILED INSTRUCTIONS FOR USE IN A PATIENT

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Let the user sit upright and adjust the height of the seat frame of lift chair with wheels so that the bottom surface of the seat frame of the lift chair with wheels is slightly higher than the user's seat plane.

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*Open the seat frame on both sides and move it to the front of the user, adjust the position of the lift chair with wheels so that the center of gravity of the user's body is in the center of the seat frame, and ***adjust the height of the seat frame** again so that the bottom of the seat frame slightly touches the user's seat plane.*

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(The photo of the product is indicative)

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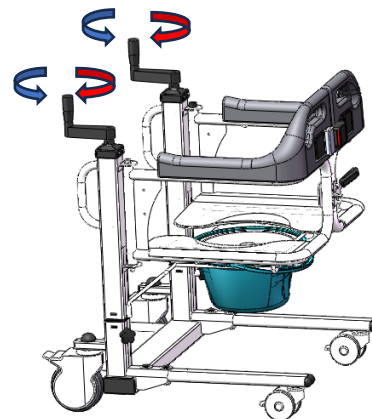
Lifting of the seat by counterclockwise rotation of the levers. 

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Seat height reduction by turning the levers clockwise. 

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
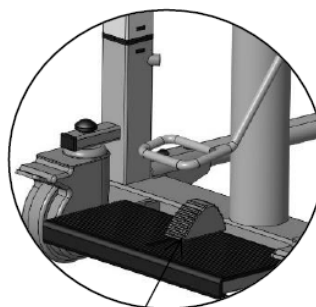
To lift the seat, place the limit block inwards (as shown in the image), and then press the hydraulic pump control system at a constant rate.

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To lower the seat height, place the limit block outwards (as shown in the image) and then press the hydraulic pump control system and hold it in this position.

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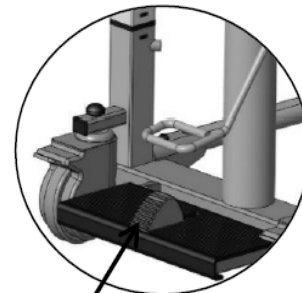Close the 5-inches front castor brake, lift the user's foot and put it in the footrest.

Let the user turn his body slightly to the left and right and at the same time, the operator rotates the corresponding side frame of the seats (left & right respectively), allowing the user to sit on the left and right frames of the lift chair with wheels.

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When the user is fully seated in the lift chair with wheels, and the left and right seat frames are fully closed, the back safety key automatically locks, and the operator locks the seat belt.

The user adjusts the comfortable posture of the body and places his hands on the armrests (right and left respectively).

The operator checks the distance between the bottom surface of the lift chair with wheels frame and the seat that the user was sitting on before, raises the seat frame 2-3 cm, releases the 5 inches front castor brake, and slowly moves the lift chair with wheels from the previous position.

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The operator pushes the lift chair with wheels from its back side to move it to the edge of the bed.

Check the height of the seat frame of the lift chair with wheels at the edge of the bed to make sure that the bottom surface of the seat frame is slightly higher than the surface of the bed (if you need to adjust the height of the seat frame, you must lock the 5 inches front castor to operate).

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The operator fully pushes the lift chair with wheels to the predetermined bed position, locks the 5-inches front castor brake, and readjusts the height of the seat frame so that its bottom surface slightly touches the bed surface.

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Open the seat belt and back safety lock at the back of the lift chair with wheels, then slowly open the left and right seat frames and allow the user to sit on the bed.

The operator unlocks the 5-inches front castor, lifts the user's feet, and moves the lift chair with wheels away from the bed.